


# VELA<sup>®</sup> ventilator quick tips

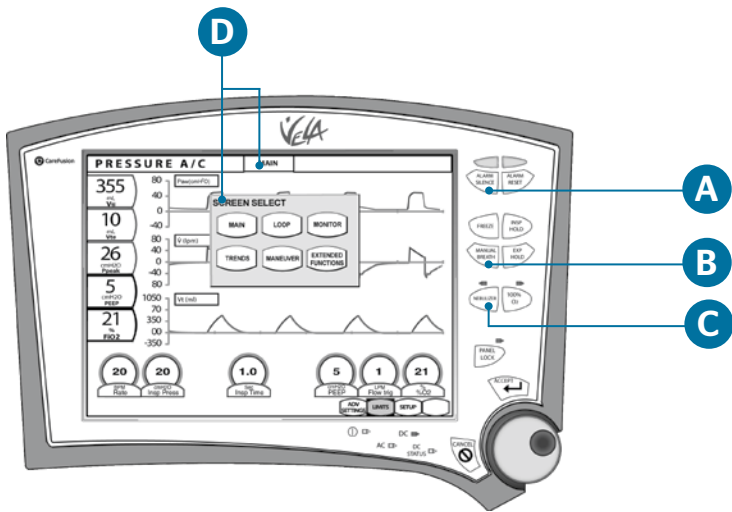

- A. Alarm silence:** 2-minute silence period.
- B. Manual breath:** Delivers a single breath at current control settings.
- C. Nebulizer:** 30 minutes, synchronized with inspiration. Delivered volumes and FiO<sub>2</sub> may be affected. Refer to operator's manual for full description.
- D. Screen indicator:** Identifies current screen. When pressed, opens the **SCREEN SELECT** menu.


The VELA<sup>®</sup> ventilator quick tips card is not intended as a replacement for the operator's manual.

You must become completely familiar with the operator's manual before using the VELA ventilator.




## Touch-Turn-Touch™ and Touch-Turn-Accept™ methods

1. Touch the control to select. The control changes color, indicating it is active.

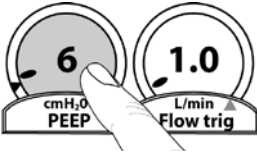

2. Turn the data dial to reach the selected value.
3. Touch the control again, or press **ACCEPT** to confirm the change.

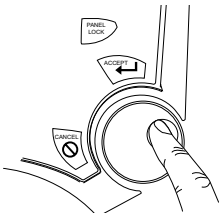

## Setting alarm limits

1. Press **LIMITS** to open or close the window.






## Setting alarm limits (continued)

- Use the Touch-Turn-Touch or Touch-Turn-Accept method to modify alarms.



## Trends

Press the screen indicator. Select **TRENDS**. Use the Touch-Turn-Touch or Touch-Turn-Accept method to configure histograms and spreadsheet columns from monitored parameters and to scale histograms. To view trends over time, press **FREEZE**. Use the data dial to scroll back in time.

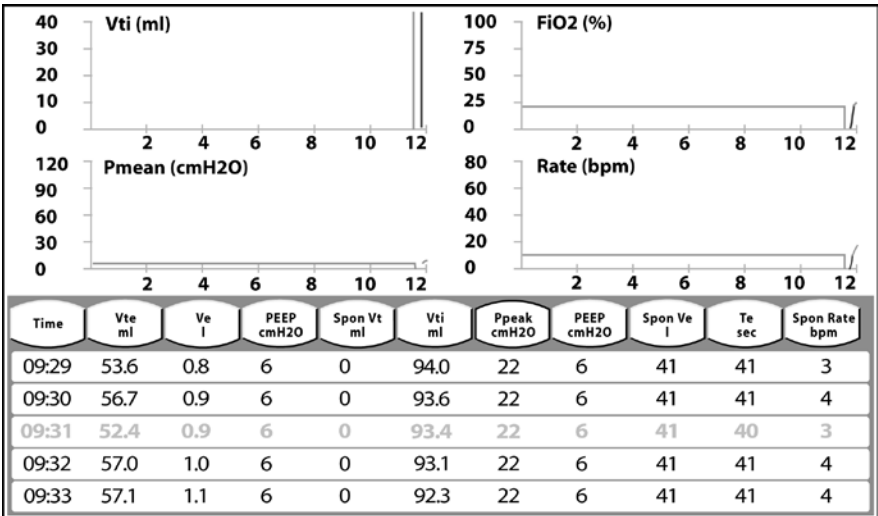



CareFusion  
22745 Savi Ranch Parkway  
Yorba Linda, CA 92887

800.231.2466 toll-free  
714.283.2228 tel  
714.283.8493 fax



CareFusion  
Yorba Linda, CA

[carefusion.com](http://carefusion.com)

© 2012 CareFusion Corporation or one of its subsidiaries. All rights reserved. Touch-Turn-Touch, Touch-Turn-Accept and VELA are trademarks or registered trademarks of CareFusion Corporation or one of its subsidiaries. RC2238-01 (0412/2000) L2292 Rev. C



**CareFusion**