

Informational Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

Information on Respiratory Diagnostics Products in context of COVID-19

Date: 2020-03-23

Originator: Respiratory Diagnostics (RDx)

Hardware related: RDx Pulmonary Function testing Products (PFT)

Background

As the Coronavirus (COVID-19) epidemic continues to escalate we would like to update our customers, partners and employees on product information, hygiene and cleaning method to minimize cross contamination using Vyairé's Pulmonary Function testing equipment (PFT).

[Safe use of PFT devices with the MicroGard™ II PFT Bacterial and Viral Filter.](#)

The use of the MicroGard II PFT Bacterial and Viral Filter is one of the effective methods minimizing cross contamination performing pulmonary function tests. Vyairé is [recommending](#) the use for [all pulmonary function tests](#). All designs of the MicroGard Filter are tested annually at NELSON Laboratories, LLC, Salt Lake City, U.S.A. with [the viral filter efficiency of 99,999% and bacterial filter efficiency of 99,999% against cross contamination](#). Please click on the statement link below to learn how this correlates to the COVID-19 virus.

[Statement \(Link\)](#) - Effectiveness of MicroGard™ II Filter in relation of COVID-19

[LINK](#) - Technical Specification MicroGard™ II Filter

Additionally, in the laboratories of Senetics Healthcare Group Germany, biological tests regarding the accumulation of bacterial population in accordance with the [DIN EN ISO 11737-1.2006](#) were conducted and successfully passed for the MicroGard II filter series.

Informational Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

Manufacturers Hygiene and Cleaning

In order to support you with a validated and tested procedure almost all PFT devices are coming with the associated Hygiene & Cleaning instruction, typically printed, electronically or both. Please navigate to the [HELP menu](#) in your SentrySuite or Vmax Software installation. The [hygiene and cleaning instructions are not limited](#) to COVID-19 virus and apply in general. If you are unable to locate the information our teams are happy to assist.

For further requests please contact our support lines.

Technical Product Support:

United States:

Toll-Free: 1.800.231.2466
Phone: 1.714.283.2228

Vyntus & Vmax Products
Email: support.vmax.us@vyaire.com

JAEGER, MasterScreen, Oxycon Products
Email: support.jaeger.us@vyaire.com

Europe, Middle East, Africa, LATAM:

Email: support.rt.eu@vyaire.com
Hotline: +49-931-4972-127

Asia Pacific, Japan, ANZ:

Email: support.rt.eu.jae@vyaire.com
Hotline: +49-931-4972-127

China:

Email: dl-cn-gcs@vyaire.com
Hotline: +86-400-878-8885