

## Subject: SentrySuite<sup>™</sup> – Windows<sup>®</sup> 10 Security Update Compatibility

Date: 2022-08-25 (last update)

Originator: Timo Kosig

### Background

Microsoft<sup>®</sup> is identifying Operating System (OS) and native Microsoft application-level vulnerabilities which impact the Windows 10 OS which is part of the environment in which Vyaire Medical's SentrySuite<sup>™</sup> software runs.

While these vulnerabilities do not impact SentrySuite software directly, changes made by Microsoft to remediate these issues may cause compatibility issues with SentrySuite.

### How Microsoft Is Responding

Microsoft regularly publishes new builds for its Windows 10 OS, called Quality Updates. The Quality Updates may contain important security patches for vulnerabilities in the Windows 10 OS. An overview of Windows 10 Quality Updates may be found here: [Windows 10 update history](#).

### How Vyaire Is Responding

Vyaire's SentrySuite software has been validated for use with current Microsoft Windows 10 versions **21H1** and **21H2**.

As Microsoft releases new Quality Updates for Windows 10 21H1 and 21H2, each version of SentrySuite compatible with Windows 10 which is not considered End-Of-Life is tested for compatibility with those new updates.

Each Quality Update contains the cumulative changes of all previously released updates. Testing of Quality Updates which are not security related may be delayed.

This document will be regularly updated to reflect the outcome of Vyaire's compatibility tests and therefore provide a guideline which patches may be installed without compromising SentrySuite operation.

# Product Security Bulletin

Important Information - Please Read and Keep



## Compatibility test results

Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
12-JUL-2022	<b>21H2:</b> 19044.1826 <b>21H1:</b> 19043.1826	<a href="#">KB5015807</a>	Yes	<b>3.20.3</b> ✓ <b>3.10.7</b> ✓ <b>3.0.10</b> ✓	
14-JUN-2022	<b>21H2:</b> 19044.1766 <b>21H1:</b> 19043.1766	<a href="#">KB5014699</a>	Yes	<b>3.20.3</b> ✓ <b>3.10.7</b> ✓ <b>3.0.10</b> ✓	
10-MAY-2022	<b>21H2:</b> 19044.1706 <b>21H1:</b> 19043.1706	<a href="#">KB5013942</a>	Yes	<b>3.20.3</b> ✓ <b>3.10.7</b> ✓ <b>3.0.10</b> ✓	
08-MAR-2022	<b>21H1:</b> 19043.1586 <b>20H2:</b> 19042.1586	<a href="#">KB5011487</a>	Yes	<b>3.20.3</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	See (7)
08-FEB-2022	<b>21H1:</b> 19043.1526 <b>20H2:</b> 19042.1526	<a href="#">KB5010342</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	
11-JAN-2022	<b>21H1:</b> 19043.1466 <b>20H2:</b> 19042.1466	<a href="#">KB5009543</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	
14-DEC-2021	<b>21H1:</b> 19043.1415 <b>20H2:</b> 19042.1415 <b>20H1:</b> 19041.1415	<a href="#">KB5008212</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	See (6)
09-NOV-2021	<b>20H2:</b> 19042.1348 <b>20H1:</b> 19041.1348	<a href="#">KB5007186</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	
12-OCT-2021	<b>20H2:</b> 19042.1288 <b>20H1:</b> 19041.1288 (see Notes)	<a href="#">KB5006670</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	See (5)
14-SEP-2021	<b>20H2:</b> 19042.1237 <b>20H1:</b> 19041.1237	<a href="#">KB5005565</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	
10-AUG-2021	<b>20H2:</b> 19042.1165 <b>20H1:</b> 19041.1165	<a href="#">KB5005033</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	
13-JUL-2021	<b>20H2:</b> 19042.1110 <b>20H1:</b> 19041.1110	<a href="#">KB5004237</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	

# Product Security Bulletin

Important Information - Please Read and Keep



Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
06-JUL-2021 (out of band)	<b>20H2:</b> 19042.1083 <b>20H1:</b> 19041.1083	<a href="#">KB5004945</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
08-JUN-2021	<b>20H2:</b> 19042.1052 <b>20H1:</b> 19041.1052	<a href="#">KB5003637</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
11-MAY-2021	<b>20H2:</b> 19042.985 <b>20H1:</b> 19041.985	<a href="#">KB5003173</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
13-APR-2021	<b>20H2:</b> 19042.928 <b>20H1:</b> 19041.928 <b>1909:</b> 18363.1500	<a href="#">KB5001330</a> <a href="#">KB5001330</a> <a href="#">KB5001337</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.4</b> ✓ <b>3.0.7</b> ✓	See (4)
09-MAR-2021	<b>20H1:</b> 19041.867 <b>1909:</b> 18363.1440	<a href="#">KB5000802</a> <a href="#">KB5000808</a>	Yes	<b>3.20</b> ✓ <b>3.10.4</b> ✓ <b>3.0.7</b> ✓	
09-FEB-2021 11-FEB-2021	<b>20H1:</b> 19041.804 <b>1909:</b> 18363.1379	<a href="#">KB4601319</a> <a href="#">KB5001028</a>	Yes	<b>3.20</b> ✓ <b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	
08-DEC-2020	<b>1909:</b> 18363.1256 <b>1903:</b> 18362.125	<a href="#">KB4592449</a>	Yes	<b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	See (3)
13-OCT-2020	<b>1909:</b> 18363.1139 <b>1903:</b> 18362.1139	<a href="#">KB4577671</a>	Yes	<b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	
16-SEP-2020	<b>1909:</b> 18363.1110 <b>1903:</b> 18362.1110	<a href="#">KB4577062</a>	No	<b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	
08-SEP-2020	<b>1909:</b> 18363.1082 <b>1903:</b> 18362.1082	<a href="#">KB4574727</a>	Yes	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
20-AUG-2020	<b>1909:</b> 18363.1049 <b>1903:</b> 18362.1049	<a href="#">KB4566116</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
11-AUG-2020	<b>1909:</b> 18363.1016 <b>1903:</b> 18362.1016	<a href="#">KB4565351</a>	Yes	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	

# Product Security Bulletin

Important Information - Please Read and Keep



Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
21-JUL-2020	<b>1909:</b> 18363.997 <b>1903:</b> 18362.997	<a href="#">KB4559004</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
14-JUL-2020	<b>1909:</b> 18363.959 <b>1903:</b> 18362.959	<a href="#">KB4565483</a>	Yes	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
16-JUN-2020	<b>1909:</b> 18363.904 <b>1903:</b> 18362.904	<a href="#">KB4567512</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
09-JUN-2020	<b>1909:</b> 18363.900 <b>1903:</b> 18362.900	<a href="#">KB4560960</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	See (2)
12-MAY-2020	<b>1909:</b> 18363.836 <b>1903:</b> 18362.836	<a href="#">KB4556799</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
21-APR-2020	<b>1903:</b> 18362.815	<a href="#">KB4550945</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
14-APR-2020	<b>1903:</b> 18362.778	<a href="#">KB4549951</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.5</b> ✓	
30-MAR-2020	<b>1903:</b> 18362.753	<a href="#">KB4554364</a>	No	<b>3.10.1</b> ✓ <b>3.0.4</b> ✓ <b>2.21.5</b> ✓	
24-MAR-2020	<b>1903:</b> 18362.752	<a href="#">KB4551762</a>	No	<b>3.10.1</b> ✓ <b>3.0.4</b> ✓ <b>2.21.5</b> ✓	
12-MAR-2020	<b>1903:</b> 18362.720	<a href="#">KB4551762</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.1</b> ✓ <b>3.0.4</b> ✓ <b>2.21.5</b> ✓	See (1)
10-MAR-2020	<b>1903:</b> 18362.719	<a href="#">KB4540673</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.1</b> ✓ <b>3.0.4</b> ✓ <b>2.21.5</b> ✓	

# Product Security Bulletin

Important Information - Please Read and Keep



Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
27-FEB-2020	1903:18362.693	<a href="#">KB4535996</a>	No	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
11-FEB-2020	1903:18362.657	<a href="#">KB4532693</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
28-JAN-2020	1903:18362.628	<a href="#">KB4532695</a>	No	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
14-JAN-2020	1903:18362.592	<a href="#">KB4528760</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	

Vyairé recommends installing any relevant security updates that have been tested as compatible per the compatibility test results table above.

## Compatibility test notes

- (1) Microsoft OneNote update is installed with this build and sets the default printer to OneNote. Default printer needs to be manually changed back to previously selected default printer. This does not affect SentrySuite.
- (2) Microsoft introduced an issue in this build which may cause issues with printing to a physical or virtual printer. Vyairé does not recommend installing this build but instead to install build 904, released by Microsoft on 16-JUN-2020 which mitigates this issue.
- (3) This will be the last Windows update released by Microsoft for Windows 10 1903. Windows 10 1903 has reached “End of Life” (EOL) status as of 08-DEC-2020.
- (4) This will be the last Windows update released by Microsoft for Windows 10 1909. Windows 10 1909 has reached “End of Life” (EOL) status as of 11-MAY-2021.
- (5) Windows 10 20H1 Build 19041.1288 has not been tested individually. 20H1 Build 19041.1348 (9-NOV-2021) includes all changes from Build 19041.1288 and has been tested as compatible.
- (6) This will be the last Windows update released by Microsoft for Windows 10 20H1. Windows 10 20H1 has reached “End of Life” (EOL) status as of 14-DEC-2021.
- (7) This will be the last Windows update released by Microsoft for Windows 10 20H2. Windows 10 20H2 has reached “End of Life” (EOL) status as of 10-MAY-2022.

## SentrySuite and Windows 10 compatibility testing plan

All Windows 10 versions released by Microsoft have a defined “End of Life” (EOL) date after which Microsoft won’t release any more quality or security updates for the version. For details and EOL dates, please see Microsoft’s [Windows 10 release information website](#) and the [Windows Lifecycle FAQ](#).

The following table explains which SentrySuite version is tested against which Windows 10 feature version:

Windows 10 Version	Windows 10 EOL date	Compatibility test status	SentrySuite Version			
			3.20	3.10	3.0	2.21 <sup>1</sup>
21 H2	13-JUN-2023	Tests ongoing	x	x	x <sup>3</sup>	
21 H1	13-DEC-2022	Tests ongoing	x	x	x	
20 H2 <sup>2</sup>	10-MAY-2022	Tests stopped 10-MAY-2022	x	x	x	
2004 (20 H1) <sup>2</sup>	14-DEC-2021	Tests stopped 14-DEC-2021	x	x	x	x
1909 (19 H2) <sup>2</sup>	11-MAY-2021	Tests stopped 13-APR-2021	x	x	x	x
1903 (19 H1) <sup>2</sup>	08-DEC-2020	Tests stopped 08-DEC-2020	--	x	x	x
1809 <sup>3</sup>	10-NOV-2020	Tests stopped	--	--	x	x
1803 <sup>3</sup>	12-NOV-2019	Tests stopped	--	--	x	x

<sup>1</sup> SentrySuite 2.21 is out of support. Tests have been stopped.

<sup>2</sup> Tests for Windows 10 security updates have been stopped since these Windows 10 versions have reached their EOL date and are not supported by Microsoft anymore.

<sup>3</sup> SentrySuite 3.0 will become EOL with the release of SentrySuite 3.30. Tests will be stopped for 3.0 once it is EOL.

Vyairé does not recommend using Windows 10 versions which have been marked as EOL by Microsoft.

These Windows 10 versions will not receive regular quality and security updates by Microsoft anymore and may be vulnerable to threats that emerged after the EOL date.

# Product Security Bulletin

Important Information – Please Read and Keep



## Generic controls

- Ensure your data has been backed up and stored according to your individual process and that your disaster recovery procedures are in place.
- Update your anti-virus and malware protection, where available.

For product or site-specific concerns, contact your Vyairé service representative.

For more information on Vyairé's proactive approach to product security and vulnerability management, contact us at [productsecurity@vyaire.com](mailto:productsecurity@vyaire.com) or visit [www.vyaire.com/product-security](http://www.vyaire.com/product-security)

A handwritten signature in black ink, appearing to read "Timo Kosig".

**Timo Kosig**  
timo.kosig@vyaire.com  
Product Security Leader  
Vyairé Medical GmbH

A handwritten signature in black ink, appearing to read "Holger Schmitt".

**Holger Schmitt**  
holger.schmitt@vyaire.com  
Senior Product Manager Software and IT integration  
Vyairé Medical GmbH

## GLOBAL HEADQUARTERS

Vyairé Medical, Inc.  
26125 North Riverwoods Blvd  
Mettawa, IL 60045  
USA